

'ACRYL-MARVEL' TECHNIQUE HINTS

DEAR NEW "ACRYL-MARVEL" USER:

"Congratulations," you have just received a product that is designed to simplify finishing procedures and improve the appearance of the final denture product. The correct use of "Acryl-Marvel" will provide a higher shine with the possibility of maintaining all special individualized characterization which may have been meticulously formed into the denture. To assist in starting with this unique, break-through product, we offer the following technique hints & suggestions:

1. 'Acryl-Marvel' is designed to remove finishing scratches and marks. All shaping and forming of the denture or appliance should be completed using a carbide bur or arbor band, etc., prior to the removal of finishing scratches with "Acryl-Marvel."

2. To totally eliminate the use of the pumicing procedure, simply finish the denture with an application of a silicone point (available from DVA inc.) or other appropriate 'smoothing rotating instruments,' keeping in mind any time spent removing scratches prior to finishing and polishing the appliance will be saved later during the final polishing process.

3. Always use a polishing buff or wheel that has not been used with other materials (pumice, etc.).

4. Apply 'Acryl-Marvel' sparingly on the polishing wheel; it is not designed to be used as if it were pumice. The material will not 'fly away' from the polishing area; it will remain on the working site and prove a very efficient scratch removal and polishing agent.

5. If a build-up of 'white appearing material' is noted on the acrylic during finishing and polishing, it is merely excess 'Acryl-Marvel' being expressed from the polishing wheel. **USE LESS MATERIAL!**

6. 'Acryl-Marvel' will instill a high-shine luster beyond what is normally expected of a polishing material; the less material used, the faster the shine will be imparted. **LESS IS BETTER!**

7. As you continue to apply less and less pressure, the appliance will gradually develop a high, lustrous shine; one that will render the final appliance extremely smooth with improved resistance to calculus build-up. Your accounts will be pleased, as will the patient. **ENJOY!**

IF YOU HAVE ANY FURTHER QUESTIONS,
PLEASE CALL OUR TOLL-FREE NUMBER 800.228.6696.

The Staff of Dental Ventures of America, Inc.

DVA dental
ventures of
america, inc.

(800) 228-6696 (951) 270-0606
www.dentalventures.com



'Repair Bits'

